IDEAL SAMPLE SHIPMENT CONDITIONS:

1. Place in the freezer the icepack that you will find in the bottom of the box for about 24 hours and make sure that you put it **frozen** back in the bottom of the box before sending the sample.
2. Ship the sample ideally on the same day of sampling.
3. During the period May - September, it is suggested to send the sample from Monday to Thursday.

SAMPLING INSTRUCTIONS

1. Blood sampling can be done at any time of the day, regardless of whether a meal has preceded it or not.
2. Write the examinee's name on the vial, without covering the code.
3. Fill the vial (10ml) completely up to the fill line.
4. Since the Cell-Free DNA BCT® vial contains chemical additives, it is important to avoid backflow from the vial.
5. Mix immediately by gently swirling the vial 8 to 10 times. Inadequate or delayed mixing may result in incorrect assay results or insufficient product yield.
6. The sample can be kept for up to 7 days, at a temperature of 6-25°C. **DO NOT FREEZE the sample.**

SHIPPING INSTRUCTIONS

1. Make sure you place the ice pack frozen in the bottom of the box and that the vial has your name on it.
2. Place the vial together with the filter paper in the special bag included in the box and close with the special adhesive tape.
3. Place the sample bag inside the box and secure the side protrusions in the corresponding recesses.
4. Place the box containing your sample into the ACS bag already included in the package.
5. Go to your nearest ACS Courier store or call ACS for pickup at your location and use the IQ Genomix Billing Code for free shipping. The billing code is written on the card placed inside the package.

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